



Scholl Complete Corn Treatment Kit

Uses

For the treatment and removal of corns. Under 16 years: not to be used except on medical advice.

Dosage

Over 16 years: ensure feet are clean and dry. Place medicated disc over centre of corn and cover with a corn plaster. Repeat treatment daily. Place cushion firmly over the centre of the corn or sore spot. Do not use for more than 2 weeks without medical advice. Under 16 years: not to be used except on medical advice.

Format and Ingredients

Pads, cushions and disc system containing:
Salicylic Acid 40% w/w,

Category:
Foot care

Manufacturer:
SCHOLL'S WELLNESS COMPANY
LTD

Pack size: 6 pads, 6 discs
and 9 foam cushions

RRP: £5.99

Legal Status: MD